Cultura

EN Instructions for Use

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For dental use only!
Made in Switzerland
Ivoclar Vivadent AG, FL-9494 Schaan/Liechtenstein

ivoclar
vivadent®
clinical

- EN Instructions for Use**
Page 3
- DE Gebrauchsinformation**
Seite 6
- FR Mode d'emploi**
Page 9
- IT Istruzioni d'uso**
Pagina 12
- ES Instrucciones de uso**
Página 15
- PT Instruções de uso**
Página 18
- SV Bruksanvisning**
Sidan 21
- DA Brugsanvisning**
Side 24
- FI Käyttöohjeet**
Sivu 27
- NO Bruksanvisning**
Side 30
- NL Gebruiksaanwijzing**
Pagina 33
- EL Οδηγίες χρήσης**
Σελίδα 36
- TR Kullanma Talimatı**
Sayfa 39
- RU Инструкция**
Стр. 42
- PL Instrukcja stosowania**
Strona 45



Safety

Intended use

Cultura is a small incubator for the incubation of yeast and bacterial cultures in the dental practice.

This apparatus has been developed solely for use in dentistry. Liability cannot be accepted for damage resulting from misuse or failure to observe the Instructions for Use. The user is responsible for testing the Cultura incubator for its use and suitability for the intended purposes. The intended use also includes the observation of the notes and regulations in these Instructions for Use.

The Cultura incubator has been shipped from the manufacturer in a safe and technically sound condition. The apparatus has been tested according to IEC 61010-1 (EN 61010-1) and EMC regulations.

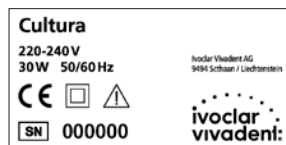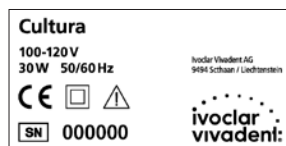Assumption of impaired safety

If it has to be assumed that safe operation is no longer possible, the power must be disconnected to avoid accidental operation. Safe operation may be impaired if, for example, the apparatus is visibly damaged or no longer works correctly. The unit is only completely disconnected from the power supply if the power cord has been pulled out of the wall socket.

Operating voltage

Before switching on the device, make sure that the voltage mentioned on the rating plate complies with your local power supply.

Observe Instructions for Use

Do not use the appliance near flammable or combustible substances.

Double insulation (apparatus complies with protection class II)

Operation

Start-up and operation

1. Setting up the unit

Set up the Cultura incubator in a horizontal position.

2. Power supply

Make sure that the voltage indicated on the rating plate complies with the local power supply.

Subsequently, connect the power cord to the power supply.

3. Switching on the unit

Switch on the unit using the ON/OFF switch (No.1).


The pilot lamp lights up.

4. Operating temperature

The preset operating temperature of approx. 35°C / 95°F is reached after approx. 30 minutes.

The operating temperature can be altered by using a screw driver to turn the setting button. Turning the button 1 mm to the right causes the temperature to increase by approx. 1°C / 1,8°F, turning it to the left causes the temperature to decrease by 1°C / 1,8°F.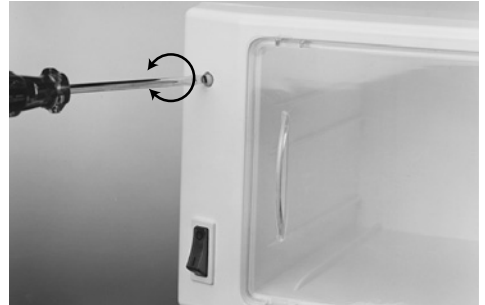

Note:

- Always keep the door tightly closed.
- Do not open the door wider than at a 90° angle (risk of fracture).
- Do not place the apparatus next to sources of external heat, such as a radiator or sunlight.
- Do not place plastic tubes directly on the heated base. Use the heat protection base supplied with the incubator.

Cleaning

The incubator is made of ABS 2002 and the doors are made of styrolux. The apparatus can be cleaned with most customary cleaning agents and disinfectants. **Important:** Do not use solvents such as acetone or other similar products!

Warranty and repair

The warranty for the Cultura incubator is two years from the date of purchase. Malfunctions resulting from faulty material or manufacturing errors are repaired free of charge during the warranty period. In case of a claim under warranty, the apparatus must be sent to the dealer or directly to Ivoclar Vivadent AG in its original packaging together with the purchase document. The warranty does not cover any damage resulting from wear under standard operating conditions, transport, incorrect operation or improper handling.

Repair work may only be carried out by a certified Ivoclar Vivadent Service Center. Use only original spare parts.

Disposal

The apparatus must not be disposed in the normal domestic waste. Dispose of old units according to the corresponding legal requirements in your country.

Product specifications

Delivery form

Cultura incubator
Thermometer
Multitrack sample holder
Power cord
Instructions for Use

Technical data

Operating voltage	220–240 V / 50/60 Hz 100–120 V / 50/60 Hz
Rated capacity	30 W / 0.2 A
Protection class II Incubation temperature	+30 °C to +45 °C ± 1 °C (+86 °F to +113 °F ± 34°F)
Thermometer display	up to +60 °C (+140 °F)
Outer dimensions	w x h x d: 310 x 155 x 168 mm
Inner dimensions	w x h x d: 220 x 120 x 150 mm
Weight	1.1 kg

Operating conditions

Temperature	+10 °C to +30 °C / +50 °F to +86 °F
Relative humidity	30% to 75%
Ambient pressure	700 hPa to 1060 hPa

Transportation and storage conditions

Temperature	–20 °C to +70 °C / –4 °F to +158 °F
Relative humidity	10% to 75%
Ambient pressure	500 hPa to 1060 hPa

This apparatus has been developed solely for use in dentistry. Set-up and operation should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damage resulting from misuse or failure to observe the Instructions for Use. The user is responsible for testing the apparatus for its use and suitability for the intended purposes.

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Ivoclar Vivadent Limited
Ivoclar Vivadent, Inc.