Indications

Table with 10 columns: Au, Pt, Pd, <1.0, S, Zn, In, Ga, Ru, Ir, Re, Other

Composition

Instructions for Use / Instrucciones de Uso / Instruções de Uso / Brugsanvisning / Brugsanvisning / Brugsanvisning / Brugsanvisning / Brugsanvisning

Harmony Soft Type 1 Rich Yellow C&B Alloy

Product information for various countries: Austria, Austria, Germany, Germany, Poland, Poland, Spain, Spain, Sweden, Sweden, Italy, Italy, United Kingdom, United Kingdom, Mexico, Mexico, New Zealand, New Zealand, Australia, Australia, USA, USA, Canada, Canada

ISO 13485 Quality Management System Certified Made in U.S.A. unless otherwise indicated Made in Canada (fabrique au Canada) Litho USA 2/05 Rev. 8/06

### INSTRUCTIONS FOR USE

EN
MODELLATION
Wax to full contour.
SPRING
Provide inlay with sprues of a suitable size. Use the direct or indirect technique being sure that the reservoir is positioned in the heat center.

MELTING AND CASTING
Use a separate carbon/ceramic crucible for each alloy. Used and new alloy must be in a ratio of 1:1. Depending on the type of casting machine, follow the manufacturers instructions for use.

INDICATIONS
Currently recommended for inlays.

CONTRAINDICATIONS
For patients with known allergy/sensitivity to any major or minor constituents of this alloy, consultation with a physician is recommended.

INTERACTIONS
Galvanic effects may occur between different alloys in the same oral environment.

SPRING METHOD
RECOMMENDATIONS
DIRECT: single crowns, inlays and onlays
INDIRECT: multiple units and multiple single crowns

INSTRUCTIONS:
1.Select a sprue with a reservoir equal to or larger than the thickest cross-section of the restoration.
2.Maintain the reservoir(s) within the heat center of the investment; keep the restoration(s) approximately 5 mm from the end of the investment and no closer than 5 mm from the sides.

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