

Stratos® 3Shape Scanner Transfer Set

The Data Transfer Set for the 3Shape laboratory scanner



Quick and
efficient in the
digital world

Stratos® 3Shape Scanner Transfer Set

The Data Transfer Set for the 3Shape laboratory scanner

With this Stratos® Scanner Transfer Set, the data can be transferred from the Stratos articulator to the 3Shape scanner easily, quickly and efficiently. This patent-pending product was developed in collaboration with Baumann Dental. Distribution is carried out by Baumann Dental. The following sets are available for the Stratos articulator:



Stratos® 3Shape Scanner Transfer Set (Baumann Dental, Item no.10034)



Adesso Split magnetic plate system for Stratos (Baumann Dental, Item no.10008)

The Stratos® 3Shape Scanner Transfer Set can be ideally implemented if there is an existing conventional connection with Adesso Split. This allows the models to be exchanged between all synchronized articulators.

This is how it works:



Mount the articulator calibration object (Item no. 680343) to the upper holder and fix in place.



Fix the transfer unit in the upper part of the Stratos articulator using the Adesso Split magnetic plate system.



Fix the lower holder to the lower part of the Stratos articulator using the Adesso Split magnetic plate system, apply adjustment adhesive to the calibration object.



Close the Stratos articulator completely and allow the adhesive to dry.



Turn the Stratos articulator over. The preparation is now complete and is ready for the 3Shape Scanner transfer.

Accessories

Baumann Dental GmbH offers various accessories for the synchronization of the Stratos articulator. These can be individually selected according to the customer's requirements.

All necessary information (Item no., price, delivery date) and ordering options can be found under www.baumann-dental.de.

You can order directly:

Baumann Dental GmbH

Frankenstrasse 25

D-75210 Keltern

Tel. +49 (0)7236 933 690

Fax +49 (0)7236 933 6999

E-mail: info@baumann-dental.de

www.baumann-dental.de

Ivoclar Vivadent AG
Benderstr. 2
9494 Schaan
Liechtenstein
Tel. +423 235 35 35
Fax +423 235 33 60
www.ivoclarvivadent.com

EN/2019-06

ivoclar
vivadent
passion vision innovation